

# MAGLUMI<sup>®</sup> CMV IgG (CLIA)

## INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the qualitative determination of CMV IgG in human serum to aid in the diagnosis of CMV infection and assess the serological status of an individual using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer (including Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 1000 Plus, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus and MAGLUMI X8).

## SUMMARY AND EXPLANATION OF THE TEST

Cytomegalovirus (CMV) (from the Greek cyto-, "cell", and megal-, "large") is a genus of viruses in the order Herpesvirales, in the family (150-200nm in diameter [hiv virus 120nm]) Herpesviridae, in the subfamily Betaherpesvirinae. Within Herpesviridae, CMV belongs to the Betaherpesvirinae subfamily, which also includes the genera Muromegalovirus and Roseolovirus (HHV-6 and HHV-7)<sup>1,2</sup>. It is related to other herpesviruses within the subfamilies of Alphaherpesvirinae that includes herpes simplex viruses (HSV)-1 and -2 and varicella-zoster virus (VZV), and the Gammaherpesvirinae subfamily that includes Epstein-Barr virus<sup>3</sup>. Humans and monkeys serve as natural hosts. There are currently eight species in this genus including the type species, human cytomegalovirus (HCMV, human herpesvirus 5, HHV-5), which is the species that infects humans. Diseases associated with HHV-5 include glandular fever, and pneumonia. In the medical literature, most mentions of CMV without further specification refer implicitly to human CMV. Human CMV is the most studied of all cytomegaloviruses<sup>3</sup>.

Cytomegalovirus (CMV) usually causes asymptomatic infection after which it remains latent in patients, primarily within bone marrow derived cells. Primary CMV infection in immunocompetent individuals may also manifest as a mononucleosis-type syndrome, similar to primary Epstein-Barr virus infection, with fever, malaise, and lymphadenopathy<sup>4</sup>. CMV is a significant cause of morbidity and mortality among bone marrow or solid organ transplant recipients, individuals with AIDS and other immunosuppressed patients due to virus reactivation or from a newly acquired infection. Infection in these patient populations can affect almost any organ and lead to multiorgan failure<sup>5,6</sup>. CMV is also responsible for congenital disease among newborns and is one of the TORCH infections (toxoplasmosis, other infections including syphilis, rubella, CMV, and herpes simplex virus). The determination of CMV IgG antibodies, can aid in the diagnosis of diseases caused by cytomegalovirus, is used to assess the serological status of an individual and is indicative for an acute or past infection, together with the detection of CMV IgM. This is particularly important in order to adopt suitable prophylaxis in susceptible individuals.

## PRINCIPLE OF THE TEST

The CMV IgG assay is an indirect chemiluminescence immunoassay.

The sample (or calibrator/control, if applicable), buffer (including goat Anti-human IgA, goat Anti-human IgM) and magnetic microbeads are mixed thoroughly and incubated, forming antibody-antigen complexes; after precipitation in a magnetic field, decant the supernatant, perform a wash cycle. Then add ABEI Label, and incubate to form sandwich complexes; after precipitation in a magnetic field, decant the supernatant, and then perform another wash cycle. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as RLU, which is proportional to the concentration of CMV IgG present in the sample (or calibrator/control, if applicable).

## KIT COMPONENTS

### Material Provided

Components	Contents	100 tests (REF: 130212005M)	50 tests (REF: 130612005M)
<b>Magnetic Microbeads</b>	Magnetic microbeads coated with CMV antigen, containing BSA, NaN <sub>3</sub> (<0.1%).	2.5 mL	2.0 mL
<b>Calibrator Low</b>	Low concentration of CMV IgG, containing bovine serum, NaN <sub>3</sub> (<0.1%).	2.5 mL	2.0 mL
<b>Calibrator High</b>	High concentration of CMV IgG, containing bovine serum, NaN <sub>3</sub> (<0.1%).	2.5 mL	2.0 mL
<b>Buffer</b>	Goat anti-human IgA, goat anti-human IgM containing BSA, NaN <sub>3</sub> (<0.1%).	22.5 mL	12.5 mL
<b>ABEI Label</b>	Mouse anti-human IgG labeled with ABEI, containing BSA, NaN <sub>3</sub> (<0.1%).	22.5 mL	12.5 mL
<b>Internal Quality Control</b>	CMV IgG, containing bovine serum, NaN <sub>3</sub> (<0.1%).	2.0 mL	2.0 mL

All reagents are provided ready-to-use.

### Accessories Required But Not Provided

#### MAGLUMI Series:

Reaction Module	REF: 630003
Starter 1+2	REF: 130299004M, 130299027M
Wash Concentrate	REF: 130299005M
Light Check	REF: 130299006M
Reaction Cup	REF: 130105000101

Please order accessories from Shenzhen New Industries Biomedical Engineering Co., Ltd. (SNIBE) or our authorized representatives.

## CALIBRATION

Traceability: This method has been standardized against the SNIBE internal reference substance.

Test of assay specific calibrators allows the RLU values to adjust the assigned master curve. Results are determined via a calibration curve which is instrument-specifically generated by 2-point calibration and a master curve (10 calibrations) provided via the reagent Radio Frequency Identification (RFID) CHIP.

Recalibration is recommended if any of the following conditions occurs:

- After each change of lots (Reagent or Starter 1+2).

- Every week and/or each time a new reagent kit is used (recommended).
- After instrument service is required.
- If controls lie outside the expected range.

## QUALITY CONTROL

Follow government regulations or accreditation requirements for quality control frequency.

Internal quality control is only applicable with MAGLUMI system. For instructions for use and target value refer to **MAGLUMI CMV IgG Quality Control Information**. User needs to judge results with their own standards and knowledge.

For detailed information about entering quality control values, refer to the operating instructions of MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer.

To monitor system performance and chart trends, commercially available quality control materials are required. Treat all quality control samples the same as patient samples. A satisfactory level of performance is achieved when analyte values obtained are within the acceptable Control Range for the system or within your range, as determined by an appropriate internal laboratory quality control scheme. If the quality control results do not fall within the Expected Values or within the laboratory's established values, do not report results. Take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the instructions for use.
- Rerun the assay with fresh quality control samples.
- If necessary, contact your local technical supporters or distributors for assistance.

## SPECIMEN COLLECTION AND PREPARATION

- Use standard sampling tubes or tubes containing separating gel. Collect blood aseptically following the universal precautions for venipuncture.
- Ensure that complete clot formation in specimens has taken place prior to centrifugation. Some specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time.
- If the specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results. Samples must be free of fibrin and other particulate matter.
- Do not use hemolyzed or grossly lipemic specimens as well as specimens containing particulate substance or exhibiting obvious microbial contamination. Inspect all specimens for bubbles, and remove bubbles before analysis for optimal results.
- Avoid repeated freezing and thawing. The serum sample can be frozen and thawed for two times. Specimens must be mixed thoroughly after thawing.
- Centrifuged specimens with a lipid layer on the top must be transferred to a sample cup or a secondary tube. Care should be taken to transfer only the clarified specimen without the lipaemic material.
- All samples (patient specimens and controls) should be tested within 3 hours when being placed on board the MAGLUMI System. Refer to the SNIBE service for more details of onboard sample storage constraints.
- Specimens removed from the separator, red blood cells or clot may be stored up to 7 days at 2-8°C, and stored up to 3 months frozen at -20°C or colder.
- Before shipping specimens, it is recommended that specimens be removed from the clot, red blood cells, or separator. When shipped, specimens should be packaged and labeled in compliance with applicable state, federal and international regulations covering the transport of clinical specimens and infectious substances. Specimens should be shipped frozen.
- The sample volume required for a single determination of CMV IgG is 10 µL.

## WARNING AND PRECAUTIONS FOR USERS

- **IVD**
- For *In Vitro* Diagnostic Use.
- Follow the package insert carefully. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

### Safety Precautions

- **CAUTION:** This product requires the handling of human specimens. It is recommended that all human sourced materials should be considered potentially infectious and handled in accordance with the 29 CFR. 1910.1030 Occupational exposure to bloodborne pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.
- It is recommended that all samples, all biological reagents and materials be considered potentially infectious and able to transmit infectious agents. They should therefore be disposed in accordance with the practices of your institution. Discard all materials in a safe and acceptable manner and in compliance with prevailing regulatory requirements.
- This product contains Sodium Azide. Dispose of contents and containers in accordance with all local, regional and national regulations.
- Refer to safety data sheets which are available on request.

### Handling Precautions

- Do not use reagent kits beyond the expiration date.
- Do not interchange reagent components from different reagents or lots.
- Prior to loading the reagent kit on the system for the first time, the reagent kit requires mixing to re-suspend magnetic microbeads that have settled during shipment.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- To avoid contamination, wear clean gloves when operating with a reagent kit and samples.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts which have no effect on assay efficacy.
- For detailed discussion of handling precautions during system operation, refer to the SNIBE service information.

## STORAGE AND STABILITY

- Sealed: Stored at 2-8°C until the expiration date.
- Opened at 2-8°C: Minimum stability is 4 weeks.
- On-board: minimum stability is 4 weeks.
- To ensure the best kit performance, it is recommended to place opened kits in the refrigerator after the end of the intraday test work. It is still possible to keep on using the kit beyond the opened or on-board period if the controls are found within the expected ranges.
- Keep upright for storage to facilitate later proper resuspension of magnetic microbeads.
- Keep away from sunlight.

## TEST PROCEDURE

### Preparation of the Reagent

- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully to guarantee magnetic microbeads of being totally homogenous prior to use.
- To ensure proper test performance, strictly adhere to the operating instructions of MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer. Each test parameter is identified via a RFID CHIP on the Reagent kit. For further information please refer to the operating instructions of

MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer.

### High-Dose Hook

Whenever samples containing extremely high antibody concentrations are tested, the saturation effect can mimic concentrations lower than real. However, a well-optimized two-step method excludes grossly underestimated results, because the analytical signals remain consistently high (saturation curve).

No false negative result due to high-dose hook effect was found with the CMV IgG assay.

## LIMITATIONS

- A skillful technique and strict adherence to the instructions are necessary to obtain reliable results.
- Bacterial contamination or heat inactivation of the specimens may affect the test results.
- A result within the expected range does not rule out the presence of disease and should be interpreted together with the patient's clinical picture and other diagnostic procedures.
- Diagnosis of a disease should not be based on the result of a single test, but should be determined in conjunction with clinical findings in association with medical judgement.
- Any therapeutical decision should also be taken on a case-by-case basis.
- Patient samples containing human anti-mouse antibodies (HAMA) may give falsely elevated or decreased values. Although HAMA-neutralizing agents are added, extremely high HAMA serum concentrations may occasionally influence results.

## RESULTS

### Calculation of Results

The analyzer automatically calculates the CMV IgG concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in AU/mL. For further information please refer to the operating instructions of MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer.

### Interpretation of Results

Results obtained with the CMV IgG assay can be interpreted as follows:

- Non-reactive: A result less than 2 AU/mL (<2 AU/mL) is considered to be negative.
- Reactive: A result greater than or equal to 2 AU/mL ( $\geq 2$  AU/mL) considered to be positive.

### NOTE:

- Consider to take a second sample, if possible, within an appropriate period of time (e.g., two weeks) to confirm levels of IgG.
- A negative test result does not completely rule out the possibility of an infection with CMV. Individuals may not exhibit any detectable IgG antibodies at the early stage of acute infection. The detection of CMV-specific IgG antibodies in a single sample indicates a previous exposure to CMV but is not always sufficient to distinguish between an acute or latent infection (irrespective of the level of the IgG antibody titer). In rare cases of primary CMV infection IgG antibody may be present before a specific IgM antibody response is detected. It is recommended that a follow-up sample be tested after 2 weeks.

Since there is no international standard material for CMV IgG yet, different IVD manufacturer have different traceability chain. Therefore results from assays of other manufacturers cannot be used interchangeably.

## PERFORMANCE CHARACTERISTICS

### Precision

Precision for the CMV IgG assay was determined as described in the CLSI EP5-A2, 2 controls and 3 human serum pools containing different concentration of analyte were assayed in duplicate at two independent runs per day for 20 testing days. The results are summarized in the following table:

Sample	Mean(AU/mL) (N=80)	Within-Run		Between-Run		Total	
		SD(AU/mL)	%CV	SD(AU/mL)	%CV	SD(AU/mL)	%CV
Negative serum	1.194	0.065	5.44	0.078	6.53	0.102	8.54
Low positive serum	3.526	0.184	5.22	0.086	2.44	0.203	5.76
High positive serum	20.931	0.837	4.00	0.475	2.27	0.963	4.60
Control 1	6.925	0.403	5.82	0.281	4.06	0.492	7.10
Control 2	19.541	0.788	4.03	0.194	0.99	0.812	4.16

### Analytical Sensitivity

<0.25 AU/mL

The Analytical Sensitivity represents the lowest analyte level that can be distinguished from zero.

### Recovery

Consider calibrator high of known concentration as a sample, dilute it by 1:2 ratio with diluents, and measure its diluted concentration for 10 times. Then calculate the expected concentration and recovery of measured concentration. The recovery should be within 90% -110%.

Expected (AU/mL)	Mean Measuring (AU/mL)	%Recovery
9.805	9.828	100.23

### Analytical specificity

Clinical CMV IgG negative samples, which contain potential cross-reactants including HAV, HBV, HCV, HIV, Syphilis, EBV IgG, CMV IgM, Rubella IgG, HSV IgG, RF, HAMA, ANA approved by commercially available CE-marked assay, were used to evaluate the cross-reactivity of CMV IgG assay. Of all the potential cross-reactants, none were found to cause false positive in the CMV IgG assay.

### Endogenous Interference

Substances up to the following concentrations did not interfere with the assay:

- Bilirubin 40 mg/dL
- Hemoglobin 1000 mg/dL
- Triglyceride 2000 mg/dL

### Drug Interference

Drugs up to the following concentrations did not interfere with the assay:

Drugs	Concentration
N-acetylcysteine	150 µg/mL

Methyldopa	25 µg/mL
Theophylline	60 µg/mL
Metformin	12 µg/mL
Isosorbide dinitrate	6 µg/mL
Rifampicin	48 µg/mL
Doxycycline	18 µg/mL
Cefoxitin	6600 µg/mL
Cyclosporine	2 µg/mL
Metronidazole	125 µg/mL
Ascorbic acid	60 µg/mL
Phenylbutazone	200 µg/mL
Aspirin	1000 µg/mL
Acetaminophen	400 µg/mL
Ibuprofen	500 µg/mL
Sodium salicylate	500 µg/mL

## REFERENCES

1. Mattes, F. M., McLaughlin, J. E., Emery, V. C., Clark, D. A., & Griffiths, P. D. (2000). Histopathological detection of owl's eye inclusions is still specific for cytomegalovirus in the era of human herpesviruses 6 and 7. *Journal of clinical pathology*, 53(8), 612-614.
2. Koichi Yamanishi; Arvin, Ann M; Gabriella Campadelli-Fiume; Edward Mocarski; Moore, Patrick; Roizman, Bernard; Whitley, Richard (2007). *Human herpesviruses: biology, therapy, and immunoprophylaxis*. Cambridge, UK: Cambridge University Press
3. Ryan KJ, Ray CG, eds. (2004). *Sherris Medical Microbiology* (4th ed.). McGraw Hill. pp. 556; 566-9
4. Söderberg-Nauclér, C., Fish, K. N., & Nelson, J. A. (1997). Reactivation of latent human cytomegalovirus by allogeneic stimulation of blood cells from healthy donors. *Cell*, 91(1), 119-126.
5. Kusne, S., Shapiro, R., & Fung, J. (1999). Prevention and treatment of cytomegalovirus infection in organ transplant recipients. *Transplant infectious disease*, 1(3), 187-203.
6. Rubin, R. H. (1999). Importance of CMV in the transplant population. *Transplant infectious disease: an official journal of the Transplantation Society*, 1, 3-7.



### Shenzhen New Industries Biomedical Engineering Co., Ltd.

No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China

Tel: +86-755-21536601

Fax: +86-755-28292740



### Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany

Tel: +49-40-2513175

Fax: +49-40-255726

## SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-8 °C)		Use-by date
	Contains sufficient for		Keep away from sunlight
	This way up		Authorized representative in the European Community
	<i>In vitro</i> diagnostic medical device		Kit components
	Catalogue number		Batch code